

\\BIC DEVELOPMENT\Development\Dr Zatorre\Mor-New-Protocol\cmrr_mbep2d_2mm_4x_bold

TA: 6:12 PM: FIX Voxel size: 2.0x2.0x2.0 mmPAT: Off Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	72
Dist. factor	0 %
Position	R0.1 P3.2 H12.5 mm
Orientation	T > C-39.0
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	1500 ms
TE	39.00 ms
Multi-band accel. factor	4
Filter	Prescan Normalize
Coil elements	HC2,4,6,7;NC2

Contrast - Common

TR	1500 ms
TE	39.00 ms
MTC	Off
Magn. preparation	None
Flip angle	50 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	240
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
Base resolution	96
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Distortion Corr.	Off
------------------	-----

Resolution - Filter Image

Prescan Normalize	On
-------------------	----

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	72
Dist. factor	0 %
Position	R0.1 P3.2 H12.5 mm
Orientation	T > C-39.0
Phase enc. dir.	A >> P
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	2.00 mm
TR	1500 ms
Multi-slice mode	Interleaved
Series	Descending
Multi-band accel. factor	4

Geometry - AutoAlign

Slice group	1
Position	R0.1 P3.2 H12.5 mm
Orientation	T > C-39.0
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	R0.1 P3.2 H12.5
R	0.1 mm
P	3.2 mm
H	12.5 mm
Initial Rotation	0.00 deg
Initial Orientation	T > C
T > C	-39.0
> S	0.0

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off

System - Adjustments

Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R0.1 P3.2 H12.5 mm
Orientation	T > C-39.0
Rotation	0.00 deg
A >> P	192 mm
R >> L	192 mm
F >> H	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.250133 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	1500 ms
Multi-band accel. factor	4

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active
Meas[15]	Active
Meas[16]	Active
Meas[17]	Active
Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off
Spatial filter	Off
Measurements	240
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	On
Contrasts	1
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.55 ms
Bandwidth	2264 Hz/Px

Sequence - Part 2

EPI factor	96
Gradient mode	Performance
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	4600 us
Single-band images	Off
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard